



sutherlandmedical

Aluminium Finger Splint

Application Instructions



1. Prior to the application of Aluminium Finger Splints, ensure that any open wounds are cleaned and dressed.
2. Select an appropriate size Aluminium Finger Splint. Guide Below.
3. Bend the Aluminium Finger Splint to the desired shape to suit the injury.
4. Secure the Aluminium Finger Splint into position using either an Elastic adhesive Bandage or Cohesive Bandage. Guide Below.

Size Chart

Code	Application	Width	Length	Packet
SMFS1	Finger & Arm Immobilisation	12mm	450mm	15
SMFS2	Finger & Arm Immobilisation	19mm	450mm	15
SMFS3	Finger & Arm Immobilisation	25mm	450mm	15

Please Note: Aluminium Fingers Splints are non autoclavable and are designed as a single patient use item only.

Sutherland Medical

8 Kingston Town Close Oakleigh Victoria 3166

P.O. Box 1194 Huntingdale Victoria 3166

Phone: 1300 664 027 Fax: 1300 664 028 Email: orders@sutherlandmedical.com.au

Or visit our website on: www.sutherlandmedical.com.au