

# Saf-T-Glove™ Flexible Light Handle Covers



- Reduces the cost of processing, sterilizing and storing light handles
- Patented design secures firmly and easily onto handle, helping to eliminate contamination and slippage risks
- Not made with natural rubber latex

Part Number	Description	Case Quantity
AM3611	Light Handle Cover, Flexible (Friction Fit)	100 Per Case
AM3612	Light Handle Cover, Flexible (Friction Fit)	200 Per Case
AM3613	Light Handle Cover, Flexible (Friction Fit)	300 Per Case

Serpentine rib reinforces the flange during application, eliminating contamination possibilities.



Dual angled flange offers easy application and holds cover in place.

## Product Application

1. While rotating, slide the light handle cover upward over the light handle adapter.
2. Leave ¼” of the light handle cover extended beyond the bottom of the light handle adapter.
3. Pat the bottom of the light handle cover with the palm of your hand to expel trapped air.
4. Ensure that the flange of the light handle cover has shrouded the flange of the light handle adapter.

Saf-T-Glove™ is a trademark of DeRoyal Industries, Inc. U.S. Patent No. 8,752,987

For more information about Saf-T-Glove, please call 1-800-251-9864 or visit [www.deroyal.com](http://www.deroyal.com)

**DeRoyal Industries, Inc.**  
 200 DeBusk Lane | Powell, TN 37849  
 1.800.DEROYAL or 865.938.7828  
[www.deroyal.com](http://www.deroyal.com) | Rev. 1/16 Reprint #0-2367



Improving Care. Improving Business.®