



sutherlandmedical

# Introducing Sacro Cinch Back Support:





**sutherlandmedical**

# Sacro Cinch Back Support

Sacro Cinch Back Supports are designed to alleviate pain and discomfort in the sacral region. By transferring pressure to the abdomen and in some instances restricting mobility these supports are effective during rehabilitation or treatment.

The Sacro Cinch Back Supports garments have been designed to give strong support to the lower lumbar and sacral area.

The back supports feature two pull straps and front closure. A cinch strap is used to adjust compression. The supports are manufactured from strong surgical elastic.

The Sacro Cinch Back Supports contain two pre shaped steel stays which are inserted posteriorly.

The complete range of Abdominal Binders are 100% Australian Made.

## Application

Measure around the patient's torso with a tape measure and select the most appropriate size based on the application guide below. Position the support around the patient with the closure at the front. Cinch and pull straps should not be closed until positioned correctly.

The Sutherland Medical label should be used as a guide to indicate the top of the support. Ensure that the support is firm and comfortable while the patient is standing as well as seated.

Code	Size	Application	Fitted Measure
SM820	Small	Universal	76 - 86cm
SM821	Medium	Universal	86 - 96cm
SM822	Large	Universal	96 - 106cm
SM823	Extra Large	Universal	106 - 116cm
SM824	Extra Extra Large	Universal	116 - 126cm

## Sutherland Medical Pty Limited

8 Kingston Town Close, Oakleigh, Victoria

Phone: 1300 664 027 Fax: 1300 664 028 Email: [orders@sutherlandmedical.com.au](mailto:orders@sutherlandmedical.com.au)

Or visit us today: [www.sutherlandmedical.com.au](http://www.sutherlandmedical.com.au)

ARTG No.133297