

# Tubular-Form SSB



## Shaped Support Bandaging Techniques and Applications

### Shaped Tubular-Form S.S.B Sizing

Place Tubular-Form sizing tape around the narrowest part of the limb to be covered. Read the size of Tubular-Form S.S.B. required (cm for shaped) to give the correct degree of support.

### CUTTING



If Tubular-Form is too long, fold in half and cut equal amounts from both ends.

### APPLYING



Pass the lower end of the bandage through the upper end and apply as a stocking - lower part first, then pulling up the upper part.

### ARM



For full arm applications, a shaped support bandage is ideal. A small cut approximately 4-5cm from the bottom should be made to accommodate the thumb.

### LEG



For full leg applications: Tubular-Form long S.S.B should be used. This exhibits graduated support, facilitating enhanced venous flow.

### BELOW KNEE



Shaped Tubular-Form can be easily applied by pulling into place.

Code	Size	Application	Box	Measurement	From
SM700	Small	Full Arm - 1/2 Leg	10	15-19 cm	Ankle / Wrist
SM701	Medium	Full Arm - 1/2 Leg	10	18-23 cm	Ankle / Wrist
SM702	Large	Full Arm - 1/2 Leg	10	22-27 cm	Ankle / Wrist
SM707	Extra Large	Full Arm - 1/2 Leg	10	26-35 cm	Ankle / Wrist
SM703	Small	Full Leg	10	15-19 cm	Ankle
SM704	Medium	Full Leg	10	18-23 cm	Ankle
SM705	Large	Full Leg	10	22-27 cm	Ankle
SM706	Extra Large	Full Leg	10	26-35 cm	Ankle

**All measurements are taken at the ankle or wrist**