



easywrap[®]

COMPRESSION MADE EASY



+ Microfine
GLOVE & TOECAP

Haddenham Healthcare Ltd
Crendon House, Crendon Industrial Park
Long Crendon, HP18 9BB
United Kingdom



www.hadhealth.com



01844 208842



sales@hadhealth.com



Haddenham easywrap is a UK patented* compression wrap system for the treatment of lymphoedema, chronic oedema and various other swelling related conditions (including venous insufficiency).

***www.easywrap.info/patents**

THREE PARTS

The easywrap compression system comes in three separate parts that can be used individually or in combination.



FOOT



LEG



THIGH

includes Knee Section


CHOICE OF TWO FABRICS

light

20-30mmHg

strong

30-40mmHg

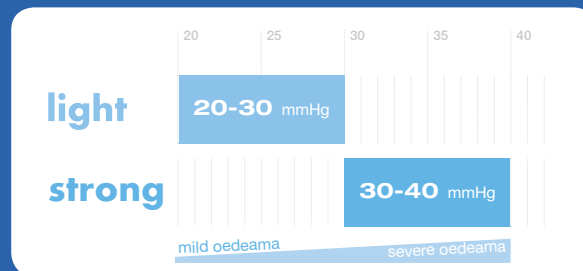

±15% + 1mmHg

easywrap garments consist of overlapping, single-layer textile bands that are worn over a protective antimicrobial liner.

The bands are secured by fasteners that are easy to attach and are engineered to be **conforming and low profile.**

easywrap's unique weave ensures bands do not neck or kink making it easier to achieve a **50% overlap** and **consistent graduated compression.**



easywrap's low profile design means that it can be worn comfortably under most every day clothing and footwear.



The mechanically limited stretch of the bands gives them an easy to feel “lock-out” or “end stretch”...


...this makes them easy to apply at the correct tension so that you can be confident that you’re getting the correct level of compression every time



**A simple and effective
alternative to bandaging
and stockings**



Can be used in combination with
Microfine® ToeCap

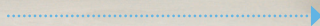




ADAPTS TO LIMB REDUCTION


easywrap's UK patented* technology provides all the benefits of classic short stretch technology whilst the elastic in the bands maintains more consistent compression as the limb reduces.

[*www.easywrap.info/patents](http://www.easywrap.info/patents)

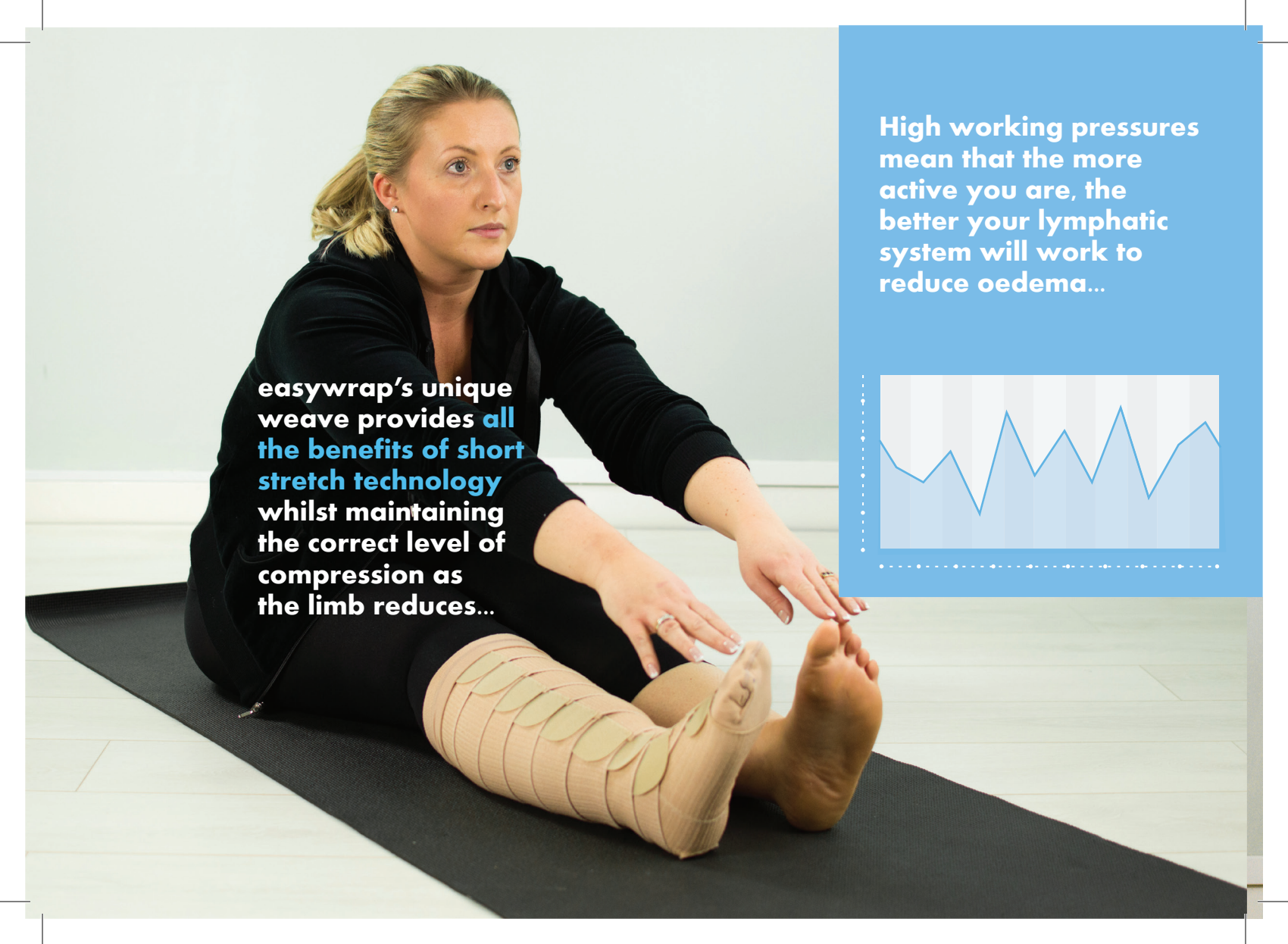


easywrap's unique weave provides **all the benefits of short stretch technology** whilst maintaining the correct level of compression as the limb reduces...

High working pressures mean that the more active you are, the better your lymphatic system will work to reduce oedema...

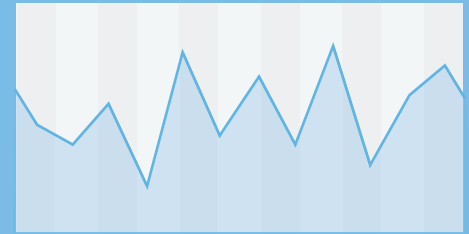

...and when you're less active easywrap's lower resting pressures maintain limb reduction.

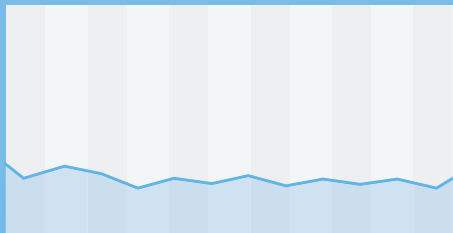

SIZING

Wide range of sizes available

MEASUREMENT POINTS

g	Just below the groin
e¹	Bottom of the thigh
d	2 finger widths below the knee crease
c	Widest point of the calf
b	Ankle
a¹	Mid foot
a	Base of toes


Take length measurements following the contour of the limb





Foot

		XS	S	M	L	XL
	a¹ Circumference	22-24	25-27	28-30	31-34	35-40
REGULAR	a-heel Length	14.5-16.5	16-18	17.5-19.5	19-21	21-23
LONG		16.5-18.5	18-20	19.5-21.5	21-23	23-25

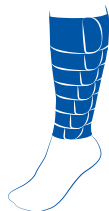

Leg

		XS	S	M	L	XL
	c Circumference	38-45	44-52	50-60	55-65	60-70
	b Circumference	23-27	28-32	33-38	39-44	45-52
REGULAR	b-d Length	30-34	30-34	30-34	30-34	30-34
TALL		34-38	34-38	34-38	34-38	34-38



Thigh with Knee

		XS	S	M	L	XL
	g Circumference	60-70	65-75	70-80	75-90	85-100
	e¹ Circumference	47-52	53-59	60-66	67-73	74-80
SHORT	e¹-g Length	16-20	16-20	16-20	16-20	16-20
REGULAR		20-24	20-24	20-24	20-24	20-24
TALL		25-30	25-30	25-30	25-30	25-30

easywrap[®] light 20-30mmHg

A more stretchy fabric suitable for mild to moderate compression needs.
Especially suitable for palliative and paediatric patients.

		Foot		Leg		Thigh <small>with Knee</small>		
		REGULAR	LONG	REGULAR	TALL	SHORT	REGULAR	TALL
STOCK	XS	EL-FR1	EL-FL1	EL-LR1	EL-LT1	EL-TS1	EL-TR1	EL-TT1
	S	EL-FR2	EL-FL2	EL-LR2	EL-LT2	EL-TS2	EL-TR2	EL-TT2
	M	EL-FR3	EL-FL3	EL-LR3	EL-LT3	EL-TS3	EL-TR3	EL-TT3
	L	EL-FR4	EL-FL4	EL-LR4	EL-LT4	EL-TS4	EL-TR4	EL-TT4
	XL	EL-FR5	EL-FL5	EL-LR5	EL-LT5	EL-TS5	EL-TR5	EL-TT5

FREE PAIR OF A-d LINERS
WHEN YOU ORDER FOOT & LEG

FREE PAIR OF A-g LINERS
WHEN YOU ORDER FOOT, LEG & THIGH

easywrap[®] strong 30-40mmHg

Similar stretch to a traditional short stretch bandage, with excellent static stiffness performance, make it suitable for patients with moderate to severe compression needs.

		Foot		Leg		Thigh <small>with Knee</small>		
		REGULAR	LONG	REGULAR	TALL	SHORT	REGULAR	TALL
STOCK	XS	ES-FR1	ES-FL1	ES-LR1	ES-LT1	ES-TS1	ES-TR1	ES-TT1
	S	ES-FR2	ES-FL2	ES-LR2	ES-LT2	ES-TS2	ES-TR2	ES-TT2
	M	ES-FR3	ES-FL3	ES-LR3	ES-LT3	ES-TS3	ES-TR3	ES-TT3
	L	ES-FR4	ES-FL4	ES-LR4	ES-LT4	ES-TS4	ES-TR4	ES-TT4
	XL	ES-FR5	ES-FL5	ES-LR5	ES-LT5	ES-TS5	ES-TR5	ES-TT5

FREE PAIR OF A-d LINERS
WHEN YOU ORDER FOOT & LEG

FREE PAIR OF A-g LINERS
WHEN YOU ORDER FOOT, LEG & THIGH



A-g

Form a barrier between the limb and easywrap which absorbs sweat and traps odours reducing the need to wash the easywrap.



A-d

antimicrobial liners

A-g LINER

Suitable for use with easywrap foot, leg and thigh garments. Please select the appropriate size for your easywrap garments.

		A-G LINER pair
STOCK	STANDARD Fits Leg Size: XS / S	EL-L-AG
	EXTRA WIDE Fits Leg Size: M / L / XL	EL-L-AGW

FREE PAIR OF A-g LINERS
WHEN YOU ORDER FOOT, LEG & THIGH

A-d LINER

Suitable for use with easywrap foot and leg garments. Please select the appropriate size for your easywrap garments.

		A-D LINER pair
STOCK	STANDARD Fits Leg Size: XS / S / M	EL-L-AD
	EXTRA WIDE Fits Leg Size: L / XL	EL-L-ADW

FREE PAIR OF A-d LINERS
WHEN YOU ORDER FOOT & LEG

UK PATENT PENDING KNEE SECTION

An easy to apply three band system. Included with easywrap THIGH (or available seperately).


V-Cuts in bands allow fine and precise application on patient

Elliptical cut with spacer foam pad increases comfort and mobility whilst reducing pinching behind the knee



KNEE MEASUREMENT POINTS

- e¹** Bottom of the thigh
- c** Widest point of the calf

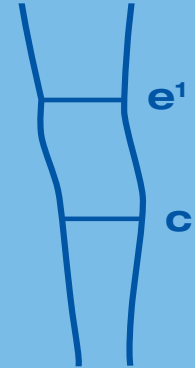

KNEE

	XS	S	M	L	XL
e¹ Circumference	47-52	53-59	60-66	67-73	74-80
c Circumference	38-45	44-52	50-60	55-65	60-70

		XS	S	M	L	XL
STOCK	LIGHT	EL-KP1	EL-KP2	EL-KP3	EL-KP4	EL-KP5
	STRONG	ES-KP1	ES-KP2	ES-KP3	ES-KP4	ES-KP5


easywrap[®] HAND 20-30mmHg

- Worn in conjunction with the Microfine[®] glove, gives the advantage of the dexterity of the Microfine[®], with the benefit of added compression to the dorsum
- Can be trimmed to fit as required, such as around the base of the fingers or around the thumb hole
- Extended wrist to prevent gapping
- Made from premium laminated fabric



		HAND LEFT	HAND RIGHT
STOCK	XS	EL-H1L	EL-H1R
	S	EL-H2L	EL-H2R
	M	EL-H3L	EL-H3R
	L	EL-H4L	EL-H4R

easywrap[®] ARM 20-30mmHg

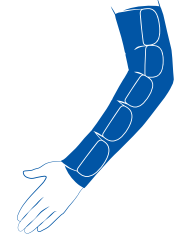

- A light weight and low profile arm wrap that can be applied and removed with one hand
- Bands can be easily tightened to accommodate reductions in swelling.
- Made from premium laminated fabric with Microfine[®] inner support sleeve


		ARM LEFT			ARM RIGHT		
		SHORT	REGULAR	LONG	SHORT	REGULAR	LONG
STOCK	S	EL-AS2L	EL-AR2L	EL-AL2L	EL-AS2R	EL-AR2R	EL-AL2R
	M	EL-AS3L	EL-AR3L	EL-AL3L	EL-AS3R	EL-AR3R	EL-AL3R
	L	EL-AS4L	EL-AR4L	EL-AL4L	EL-AS4R	EL-AR4R	EL-AL4R


SIZING

Wide range of sizes available

MEASUREMENT POINTS

- | | |
|-------------------|---|
| <i>g</i> | Axilla |
| <i>e</i> | Elbow Crease |
| <i>c</i> | Wrist Crease |
| <i>b</i> | Palm at fold of thumb |
| <i>a</i> | Palm at base of fingers |
| <i>c-g</i> | Measure from the outside of the wrist to the axilla with a slight bend in the arm |

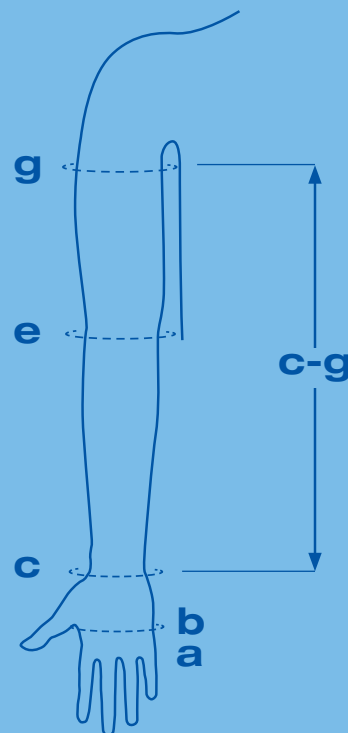

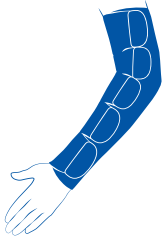

ARM

		S	M	L
c Circumference		14-18	16-21	19-25
e Circumference		20-27	25-34	30-40
g Circumference		22-31	29-39	32-45
SHORT	c-g Length	40-44	40-44	40-44
REGULAR		44-48	44-48	44-48
TALL		48-52	48-52	48-52



HAND

	XS	S	M	L
a Circumference	<18	18-20	20-22	>22
b Circumference	<19	19-21	21-23	>24



Microfine **TOECAP**

An ultra thin and smooth fabric that trims to fit without fraying

Features

- Can be used under bandaging
- No need to bandage toes
- Discreet and flexible
- Ambidextrous
- 20-36mmHg
- Easy to fit
- Easy to measure



			
		SAND	BLACK
STOCK	X-SMALL	218S-1	218B-1
	SMALL	218S-2	218B-2
	MEDIUM	218S-3	218B-3
	LARGE	218S-4	218B-4
	X-LARGE	218S-5	218B-5

Select Size

Circumferences cm		XS	S	M	L	XL
a	BALL OF FOOT	21-23	23-25	25-27	27-29	29-31



Microfine GLOVE

An ultra thin and smooth fabric that trims to fit without fraying

Features

- Can be used under bandaging
- No need to bandage fingers
- Discreet and flexible
- Ambidextrous
- No loss of dexterity
- 20-36mmHg
- Easy to fit
- Easy to measure



			
		SAND	BLACK
STOCK	XX-SMALL	208S-0	208B-0
	X-SMALL	208S-1	208B-1
	SMALL	208S-2	208B-2
	MEDIUM	208S-3	208B-3
	LARGE	208S-4	208B-4

Select Size

Circumferences cm		XXS	XS	S	M	L
a	PALM AT BASE OF FINGERS	14-16	16-18	18-20	20-22	22-24
b	PALM AT FOLD OF THUMB	15-17	17-19	19-21	21-23	24-26



www.easywrap.info

Haddenham Healthcare Ltd
Crendon House, Crendon Industrial Park
Long Crendon, HP18 9BB
United Kingdom



www.hadhealth.com



01844 208842



sales@hadhealth.com

easywrap