


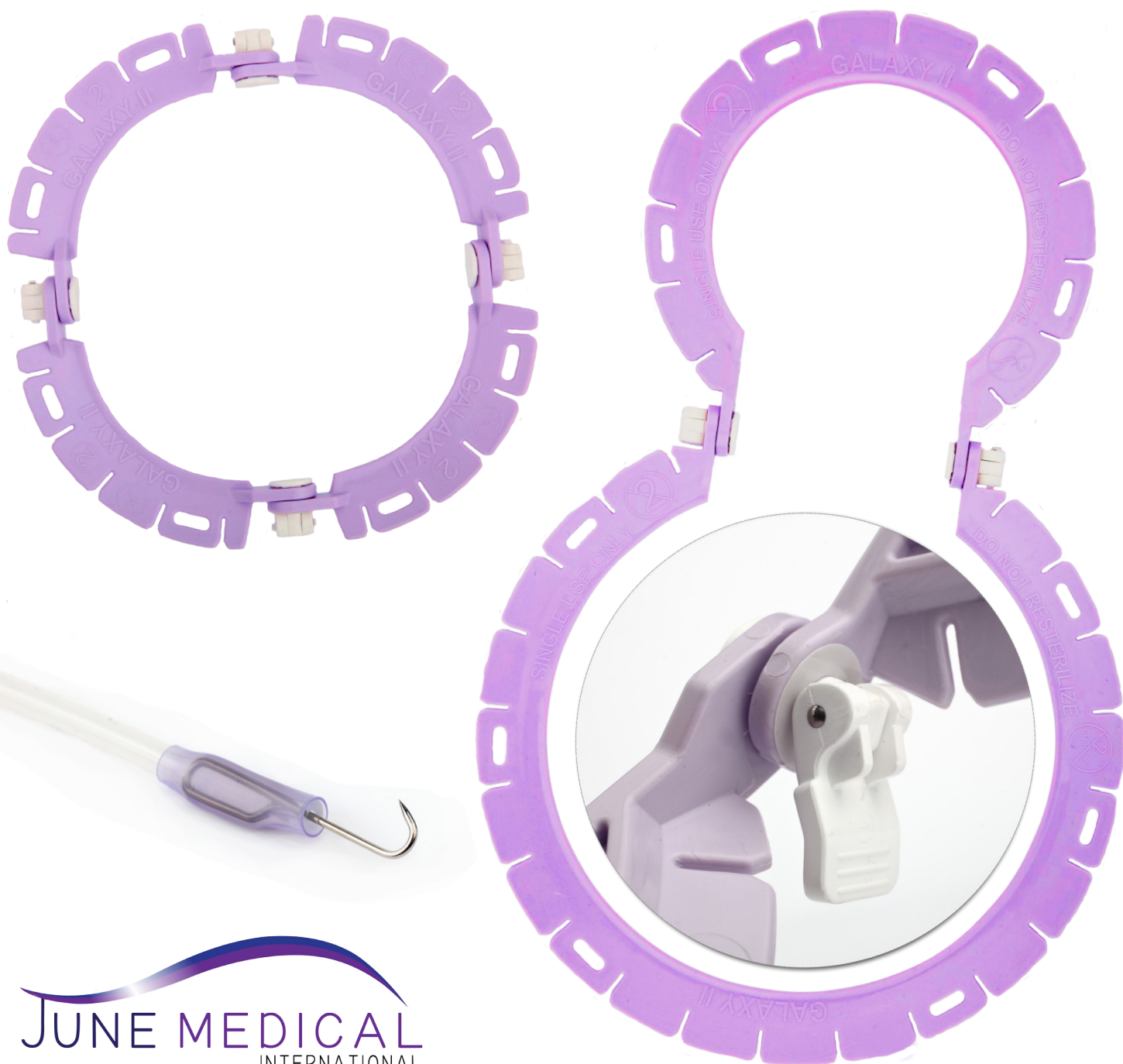

# New & Improved Sterile Surgical Retractor

**10% Less Plastic! Improved design for efficiency!**

A self-retaining, single use, retractor system providing clear visualisation, optimal exposure and access to the operative area.





Allowing for optimal angles of retraction providing better clarity and ease of use.

The stay hooks are placed into the tissue and the elastic side is inserted into the slots on the ring, providing counter traction.

The slots on the ring easily secure the stays in place, but readily release it when rapid reposition is required.

## Benefits of using a retractor


---

- Provides clear visualisation, optimal exposure and access to the operative area during surgical procedures. Reducing instrument clutter at the surgical site and the number of skilled assistants required to assist retraction
- Light-weight and intuitive to use, the surgeon inspired CAM locking system enables users to swiftly adapt the retractor to the needs of the procedure.
- Two retractor shapes to accommodate a vast range of surgical procedures
- **The silicone stays are latex free and are available as:**  
5mm blunt stays, 5mm sharp stays, 12mm blunt stays



## Instructions for Use

---

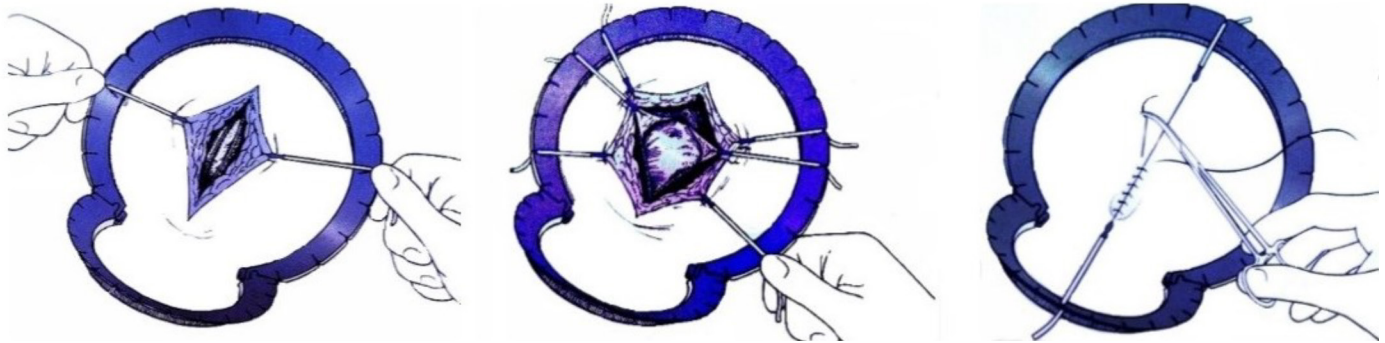

Place Galaxy around surgery site. Connect stays in directly opposing quadrants with equal tension to hold the retractor in place.

As dissection progresses, you should advance the position of the stays deeper into the tissue. Add or reposition stays to desired angle for a balanced retraction.

You need to maintain constant, opposing tension of the stays while suturing.

---


### What does the retractor system kit contain?

- A sterile disposable retractor ring (Snowman or Square)
- 8 sterile elastic stays (5mm sharp / 5mm blunt / 12 mm blunt)
- 2 x Catheter clip

**(Additional boxes of stays are also available)**

### Galaxy II is used in:

- Urology
- Gynaecology
- Colorectal
- Tracheotomy
- General Surgery
- Plastic Surgery
- Paediatric Surgery
- Laparoscopic Surgery

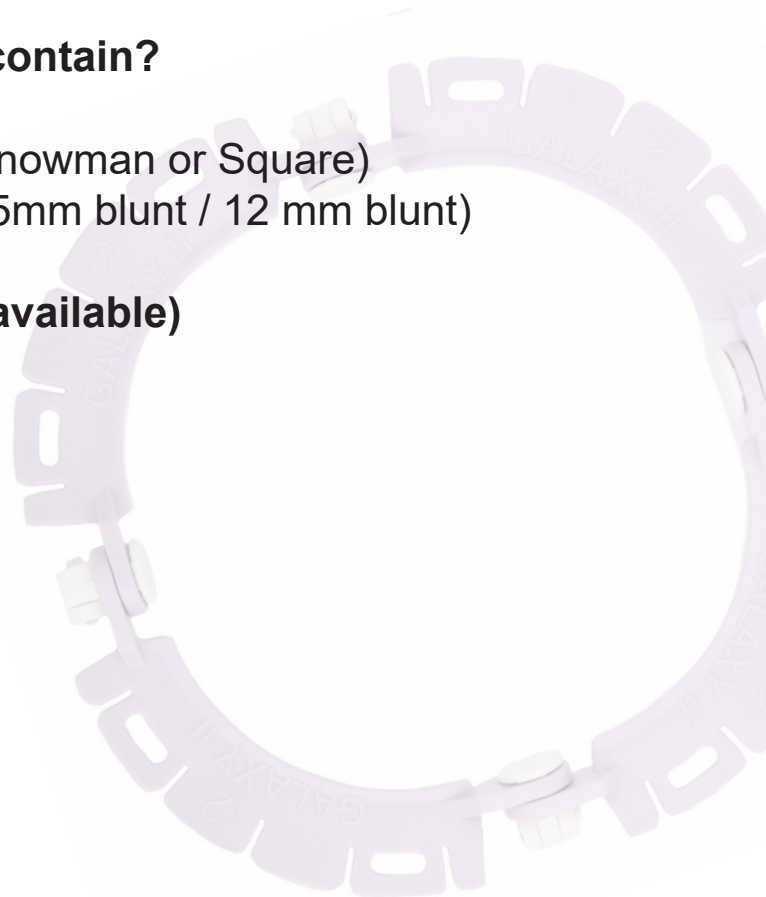



**To order contact us on:**

**[Enter your Details Here]**

Product	Description	Order Code
<b>Galaxy II Square Kit</b>	Sterile Single Use, Square Retractor System, 5mm Sharp stays x 8 & catheter clips x 2 (Box of 10)	JUNE3001S5
<b>Galaxy II Square Kit</b>	Sterile Single Use, Square Retractor System, 5mm Blunt stays x 8 & catheter clips x 2 (Box of 10)	JUNE3001B5
<b>Galaxy II Square Kit</b>	Sterile Single Use, Square Retractor System, 12mm Blunt stays x 8 & catheter clips x 2 (Box of 10)	JUNE3001B12
<b>Galaxy II Snowman Kit</b>	Sterile Single Use, Snowman Retractor System, 5mm Sharp stays x 8 & catheter clips x 2 (Box of 10)	JUNE3000S5
<b>Galaxy II Snowman Kit</b>	Sterile Single Use, Snowman Retractor System, 5mm Blunt stays x 8 & catheter clips x 2 (Box of 10)	JUNE3000B5
<b>Galaxy II Snowman Kit</b>	Sterile Single Use, Snowman Retractor System, 12mm Blunt stays x 8 & catheter clips x 2 (Box of 10)	JUNE3000B12
<b>Stays (Blunt)</b>	Sterile Individually Packaged 5 mm Blunt Stays (Pack of 100)	JUNE3002B-100
<b>Stays (Blunt)</b>	Sterile Individually Packaged 12 mm Blunt Stays (Pack of 100)	JUNE3003B-100
<b>Stays (Sharp)</b>	Sterile Individually Packaged 5 mm Sharp Stays (Pack of 100)	JUNE3002S-100



**Website:** [www.junemedical.com/international](http://www.junemedical.com/international)

**Email:** [international@junemedical.com](mailto:international@junemedical.com)

**Tel:** +44 (0) 1494 303100